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Type of product:

Ready-to-use spray disinfectant for medical devices, operating tables and other medical inventory (including acrylic glass). Broad microbiological efficacy with practically relevant contact times. Very good material compatibility. Quick, residue free drying. Bactericidal, fungicidal, limited virucidal (BVDV(HCV)/Vaccinia incl. HBV/HIV). Effective against all enveloped viruses according to the recommendation of the Robert Koch Institute.

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Certification by University and Scientific Institutes expert opinions:

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Bactericidal, fungicidal, tuberculocidal efficacy: 100% - 1 min

Hygiene Nord GmbH, Greifswald, Germany

17. August 2006

- Escherichia Coli NCTC 11229
- Pseudomonas aeruginosa ATCC 15442
- Proteus mirabilis ATCC 14153
- Enterococcus hirae ATCC 10541
- Candida albicans ATCC 10231
- Staphylococcus aureus ATCC 6538
- Mycobacterium Terrae ATCC 15755

Bovine Viral Diarrhea Virus (BVDV) efficacy (surrogate for HCV): 100% - 30 sec.

Dr. J. Steinmann, Bremen, Germany 6.

November 2006

Vaccinia strain Elstree efficacy: 100% - 30 secretary.

Dr. Jochen Steinmann, Bremen, Germany 6.

November 2006

Quantitative suspension test with high soil load, according to the standard methods of the DGHM: 100% - 1 min

Prof. Dr. med. H.- P. Werner, Schwerin, Germany 20. September 2006

- Staphylococcus aureus ATCC 6538
- Enterococcus hirae ATCC 10541
- Pseudomonas aeruginosa ATCC 15442
- Candida albicans ATCC 10231

Listed in the disinfection list of the German Society of Hygiene and Microbiology (DGHM/VAH).

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Surfaces and instruments must be completely moistened. Contact time: 1 min.

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Storage conditions: do not allow temperatures below 0°C or above 30°C;

do the product not expose to direct sunlight; keep the store dry.

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